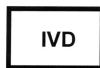
Instructions For Use

REF: LPS 018

FGFR1 Breakpart/Amplification

FOR PROFESSIONAL USE ONLY

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Further information available at www.cytocell.com

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation is available for annealing to a similarly denatured, fluorescently labelled DNA probe which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

Current methodologies, namely immunohistochemistry or Southern blotting can provide information at the gene expression level but, when carried out on tissue sections (either cryostat or paraffin-embedded), FISH can provide information at a gene level, *in situ*, at the precise site within the tumour. This can reveal cell-to-cell heterogeneity and enable the detection of small clones of genetically distinct cells.

Probe Information

FGFR1 (Fibroblast Growth Factor Receptor 1) is a protein coding gene whose largest transcript features 18 exons, measures 5.6kb and is located on chromosome 8. FGFR1 was the first gene shown to be amplified in breast cancer¹ which is the case for around 10% of such tumours.² Amplification of the gene has been shown to provide a poor prognosis as it leads to early relapse because of the over expression of the amplified gene³.

Within the sequence of the gene is an open reading frame that encodes for a putative tyrosine kinase⁴ which is likely to be the causative factor within a number of stereotypic translocations which each give rise to a recognisable leukaemic syndrome called 8p11 myeloproliferative syndrome⁵. EMS or 8p11 myeloproliferative syndrome is often also referred to as Stem cell leukaemia/lymphoma syndrome (SCLL) and is a rare aggressive condition which usually features a BCR/ABL negative myeloproliferative disorder and a precursor T-cell lymphoblastic lymphoma. Patients often develop Acute Myeloid Leukaemia⁶ which is resistant to chemotherapy. There are at least 8 partner genes involved with FGFR1 in the disorder including: ZNF198 on 13q11-12 (the most common⁷), FOP on 6q27, CEP110 at 9q33, BCR at 22q11, HERV-K at 19q13, FGFR10P2 at 12p11, TIF1 at 7q34 and MYO18A at 17q23⁸. The breakpoints have been established for the ZNF198 translocation (t(8;13)(p11;q12)) as being in exon 8 of FGFR1 and result in fusing zinc finger proteins to the tyrosine kinase domain of FGFR1 and activating the tyrosine kinase activity, similar to the mechanism behind the BCR/ABL fusion of CML.⁸ This important gene is therefore involved in both breast cancer and a myeloproliferative disorder.

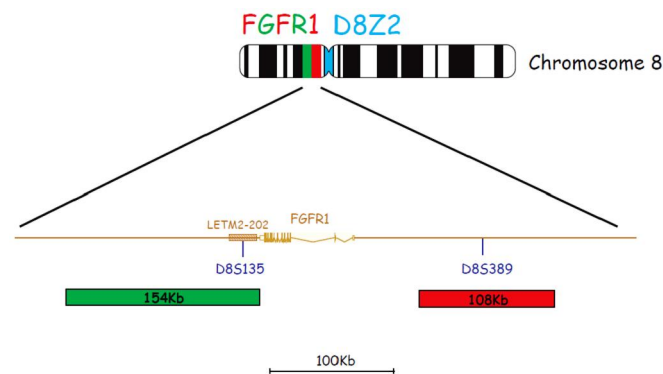
Cytocell has developed a three-colour combined breakpart and amplification FISH probe for FGFR1 which can be used in either bone marrow samples from EMS patients or tissue section in possibly amplified patients.

As it is the involvement of the FGFR1 tyrosine kinase domain that is important in EMS, detection of the partner gene involved is less important than establishing that there has been a translocation which has disrupted the FGFR1 gene. In these cases, the breakpart strategy will show the conventional split of one of the two fusion signals into a red and a green signal, along with a blue 8 centromere to enumerate chromosome 8. Although not strictly relevant in these cases, this additional probe may act as an extra enumerator in some cases.

In patients that are being tested for amplifications of FGFR1, the breakpart strategy is not relevant, but the un-translocated FGFR1 FISH signal will appear as a fusion probe and it is this fusion signal that will appear amplified. In order to properly enumerate this probe in tissue sections, the chromosome 8 centromere probe, included in a contrasting colour (blue), is useful.

Probe Specification

FGFR1 proximal, 8p11.2, Red
FGFR1 distal, 8p11.2, Green
8cen Blue

The FGFR1 probe mix consists of a green 154kb probe telomeric to the FGFR1 gene spanning the LETM2-202 gene and D8S135 marker. The red probe is 52kb centromeric to the FGFR1 gene spanning a 108kb region including the D8S389 marker.

Materials Provided

Probe: 100µl per vial (10 tests)

Amount of red FGFR1 probe: 110-138ng/test

Amount of green FGFR1 probe: 420-525ng/test

Amount of blue 8cen probe: 56-70ng/test

The probes are provided premixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use.

Counterstain: 150µl per vial (15 tests)

The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

Storage and Handling

The Aquarius kit should be stored at -20°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

Fluorescence Microscope Recommendations

For optimal visualisation of the probe we recommend a 100 watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing red and green fluorophores simultaneously. The blue fluorophore has specificity to the Aqua or DEAC spectrum (Aqua filter is required).

Sample Preparation

This kit is designed for use on cultured peripheral blood cells / cultured bone marrow cells fixed in Carnoy's fixative or paraffin-embedded tissue sections / tissue microarrays (TMA) which should be prepared according to the laboratory or institution guidelines.

For FISH on tissue sections 4-6 m thick formalin-fixed and paraffin-embedded tissue specimens should be used.

After mounting tissue section on slides, slides should be baked at 56°C overnight.

Paraffin-Embedded Tissue Sections FISH Suggested Protocol

Paraffin-embedded tissue sections require pre-treatment before hybridising with FISH probes i.e. removal of paraffin wax (de-paraffinisation), rehydration, acid treatment, pre-treatment and protein digestion.

Reagents:

1. Hydrochloric acid (HCl)
2. 2xSSC (saline sodium citrate) wash buffer

3. Histoclear (e.g. Fisher Scientific (Prod. Code: HIS-010-010S))
4. 70%, 85%, 100% Ethanol
5. Lyophilized pepsin (e.g. SIGMA cat. number: P6887, 3200-4500 U/mg)
6. Paraffin-embedded tissue sections (PETS)
7. Sodium thiocyanate NaSCN (e.g. 81.07g/mol SIGMA cat. number:251410)
8. 0.4xSSC pH 7.0 wash buffer
9. 2xSSC-0.05%Tween-20 pH 7.0 wash buffer

Reagent Preparation

1. Acid Pre-Treatment Solution
 - a. Prepare 100ml of 1M NaSCN Pre-treatment solution by dissolving 8.1g in 100ml distilled water.
 - b. Pour Pre-Treatment solution into 100ml ceramic or glass Coplin jar. Place into water bath and set the temperature at 80°C. Ensure that the temperature of the Pre-Treatment solution is equilibrated to 80°C before commencing the relevant steps.
2. 2xSSC Wash Buffer
 - a. 2xSSC Buffer should be allowed to reach room temperature before use.
3. 0.2% Pepsin in 0.01M HCl
 - a. Add 10ml of distilled water to 1g of pepsin, mix well and aliquot into five tubes (2ml per tube). Prepared 10% pepsin stock should be stored at -20°C.
 - b. Add 10ml of 0.1M HCl to 88ml of distilled water and preheat in a 37°C waterbath. 2ml of 10% pepsin stock should be added and mixed **just before** immersing slides in the solution.

De-paraffinisation of sections (Fume Cabinet)

1. Mark probe target area with a diamond marker pen.
2. Pour 100ml of Histoclear into three ceramic or glass Coplin jars, labeled %a+, %b+ and %c+. Leave the slides in jar %a+ for 10mins. Repeat with jars %b+ and %c+.
3. Remove slides from jar %a+ and remove excess solution by gently tapping the edges of each slide on a paper towel.
4. Pour the used Histoclear into waste bottles marked Histoclear %a+, %b+, %c+ when finished. Change the Histoclear after 200 slides have been processed or if cloudy in appearance.

Re-hydration

1. Place slides in a plastic jar containing 50ml of 100% ethanol (room temperature) for 2mins. Pour off the ethanol into a suitable waste container.
2. Replace with fresh 100% ethanol. After 2mins pour off the ethanol into a suitable waste container.
3. Place slides in a plastic jar containing 50ml of 70% ethanol (room temperature) for 2mins. Pour off the ethanol into a suitable waste container.
4. Replace with fresh 70% ethanol. After 2mins pour off the ethanol into a suitable waste container.
5. Air dry slides.

Acid treatment

1. Place slides in 100ml of 0.1 M HCl for 20mins.
2. Place slides in distilled water for 3mins.
3. Place slides in 2xSSC buffer for 3mins.

Pre-Treatment (Fume Cabinet)

1. Place slides in NaSCN Pre-Treatment solution pre-heated to 80°C in a waterbath and incubate for 30mins.
2. Remove slides from the jar and remove excess solution by gently tapping the edges of each slide on a paper towel.
3. Place slides in distilled water for 1min at room temperature (in a plastic jar).
4. Remove slides from the jar and place in 2xSSC Buffer for 3mins at room temperature. Pour off the buffer into a suitable waste container.
5. Replace with fresh 2xSSC buffer. After 3mins pour off the buffer into a suitable waste container.
6. Remove slides from the jar and remove the excess solution by gently tapping the edges of each slide on a paper towel.

Pepsin Digestion

1. Place slides in the pre-heated pepsin solution and incubate for 10-30mins. (Coplin jar).
2. Remove slides from the jar and gently tap the edges of each slide on a paper towel to remove the Pepsin.
3. Air dry slides.
4. Dehydrate tissue sections through the ethanol series (70%, 85% and 100%) in Coplin jars (room temperature) immersing the slides for 2mins in each.

Hybridisation

1. Add 10µl of probe per section (the amount of probe depends on a size of a section, add more if needed).
2. Apply a coverslip and seal with rubber solution glue.
3. Place the slide on a hot plate and denaturate at **75°C for 5mins**.
4. Place the slides in the 37°C incubator overnight.

Post-Hybridisation Washes

1. Remove coverslip and all traces of glue carefully.
2. Wash the slide in 0.4xSSC (pH 7.0) at 72°C for 2mins.
3. Drain the slide and wash in 2xSSC/0.05% Tween (pH 7.0) at RT for 30secs.
4. Drain the slide and apply 10-15 I ES DAPI antifade onto each sample.
5. Cover with the coverslip, remove bubbles and allow colour to develop in the dark for 10mins.
6. View with fluorescence microscope.

Comments

Hybridisation efficiency and tissue morphology are usually negatively correlated. Aggressive pretreatment procedures improving hybridisation efficiency (e.g. long incubation times of NaSCN or pepsin digestion) tend to destroy cell structure and tissue morphology. However, mild pretreatment saving tissue structures may not be sufficient for probe penetration and successful FISH results.

The optimal length of protease treatment time will depend on the age of the block, the tissue composition, and quality of tissue fixation. Protease treatment should be decreased for core biopsies and any sections that contain few tumour cells or have large areas of necrosis. These samples need to be handled particularly carefully to avoid over-digestion.

Lymphocyte or Bone Marrow FISH Suggested Protocol FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights should be limited at all times)

Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse slide in 2xSSC for 2mins at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2mins at RT.
4. Allow to dry.

Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.
7. Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to -20°C.
8. Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5mins.
9. Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

Denaturation

10. Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2mins.

Hybridisation

11. Place slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

Post-Hybridisation Washes

12. Remove the coverslip and all traces of glue carefully.
13. Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2mins without agitation.
14. Drain the slide and immerse in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30secs without agitation.
15. Drain the slide and apply 10µl of DAPI antifade onto each sample.
16. Cover with a coverslip, remove bubbles and allow colour to develop in the dark for 10mins.
17. View with a fluorescence microscope.

Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at or below room temperature.

Procedural Recommendations

1. Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by Cytocell Ltd.
2. The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
3. The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
4. Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

Expected Results

In a normal cell due to the close juxtaposition of the probes, 2 yellow fusion and 2 blue control signals will be observed (2Y, 2B). In the event of a 8p11 translocation one split and one fused signal will be observed resulting in 1 yellow, 1 red, 1 green, 2 blue conformation (1Y, 1R, 1G, 2B). In the event of an amplification of the FGFR1 gene 3 or more yellow fusion and 2 blue control signals will be observed (3+Y, 2B).

Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to classic cytogenetics and therapeutic action should not be initiated on the basis of the FISH result alone.

Customer Support

Please contact the Cytocell Sales and Marketing Department by telephone or email probes@cytocell.com.

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Reference/Bibliographie/Literatur/Bibliografía

1. Theillet,C., Adelaide,J., Louason,G. *et al.* FGFR1 and PLAT genes and DNA amplification at 8p12 in breast and ovarian cancers. *Genes Chromosomes and Cancer* 1993;7:219-26
2. Courjal,F., Cuny,M., Simony-Lafontaine, J., *et al.* Mapping of DNA amplifications at 15 chromosomal locations in 1875 Breast tumours: definition of phenotypic groups. *Cancer Research* 1997;57:4360-7
3. Turner,N., Pearson,A., Sharpe,R., *et al.* FGFR1 amplification drives endocrine therapy resistance and is a therapeutic target in breast cancer. *Cancer Res.* 2010 March 1; 70(5): 2085. 2094.
4. Ruta, M., Howk, R., Ricca, G., *et al.* A novel protein tyrosine kinase gene whose expression is modulated during endothelial cell differentiation. *Oncogene* 3: 9-15, 1988.
5. Macdonald, D., Aguiar, R. C., Mason, *et al.* A new myeloproliferative disorder associated with chromosomal translocations involving 8p11: a review. *Leukemia* 9: 1628-1630, 1995.
6. Goradia, A., Bayerl, M. and Cornfield, D. The 8p11 myeloproliferative syndrome: review of literature and an illustrative case report. *Int. J. Clin. Exp. Path.* 1: 448-456, 2008.
7. Xiao, S., Nalabolu, S. R., Aster, J. C., *et al.* FGFR1 is fused with a novel zinc-finger gene, ZNF198, in the t(8;13) leukaemia/lymphoma syndrome. *Nature Genet.* 18: 84-87, 1998.
8. Smedley, D., Hamoudi, R., Clark, J., *et al.* The t(8;13)(p11;q11-12) rearrangement associated with an atypical myeloproliferative disorder fuses the fibroblast growth factor receptor 1 gene to a novel gene RAMP. *Hum. Molec. Genet.* 7: 637-642, 1998.

Patents and Trademarks

Aquarius and Cytocell are registered trademarks of Cytocell Ltd.

Cytocell Ltd.
4 Technopark
Newmarket Road
Cambridge, CB5 8PB, UK.
T: +44(0)1223 294048
F: +44(0)1223 294986
E: probes@cytocell.com
W: www.cytocell.com