Instructions For Use

REF: LPE 0XYc / LPE 0XYq

Chromosome X α -satellite (DXZ1) and Chromosome Y α -satellite (DYZ3) dual labelled and Chromosome X α -satellite (DXZ1) and Chromosome Y satellite III (DYZ1) dual labeled

FOR PROFESSIONAL USE ONLY

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Further information available at www.cytocell.com

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei of fixed cultured or uncultured cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Target DNA, after fixation and denaturation is available for annealing to a similarly denatured, fluorescently labelled DNA probe which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed by a series of rapid formamide-free stringent washes and the DNA counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

Probe Specification

XYc
X centromere Xp11.1-q11.1 (DXZ1) Green
Y centromere Yp11.1-q11.1 (DYZ3) Red

XYq
X centromere Xp11.1-q11.1 (DXZ1) Green
Y heterochromatin Yq12 (DYZ1) Red

Introduction

Chromosome X α -satellite (DXZ1) and Chromosome Y α -satellite (DYZ3)
This enumeration probe set contains chromosome specific alphoid DNA repeat sequences located at the centromeres of chromosome X and chromosome Y.

This probe set will enable the identification and enumeration of X and Y human chromosomes in interphase cells and metaphase chromosomes from fixed cultured peripheral blood cells.

Chromosome X α -satellite (DXZ1) and Chromosome Y satellite III (DYZ1)

This enumeration probe set contains chromosome specific DNA repeat sequences located at the centromere of chromosome X and in the heterochromatic block of chromosome Y.

It will enable the identification and enumeration of X and Y human chromosomes in interphase cells and metaphase chromosomes from fixed cultured peripheral blood and bone marrow samples.

Materials Provided

Probe: 100 μ l per vial (10 tests)
XYc
Amount of X α -satellite probe: 0.6 ng/test
Amount of Y α -satellite probe: 500 ng/test

XYq
Amount of X α -satellite probe: 0.6 ng/test
Amount of Y satellite III probe: 8.3 ng/test

The XY probes are provided premixed and ready to use in hybridisation solution (Formamide; Dextran Sulphate; SSC). The probe DNA is directly labelled: X with a green fluorophore (FITC spectrum) and Y probes with a red fluorophore (Texas Red spectrum).

Counterstain: 150 μ l per vial (15 tests)

The counterstain is DAPI antifade (ES: 0.125 μ g/ml DAPI (4,6-diamidino-2-phenylindole))

Warnings and Precautions

- For *in vitro* diagnostic use. For professional use only.
- Wear gloves when handling DNA probes and DAPI counterstain.
- Probe mixtures contain formaldehyde which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
- DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
- All hazardous materials should be disposed of according to your institutions guidelines for hazardous waste disposal.

Storage and Handling

The Aquarius kit should be stored at -20°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

Equipment Necessary but not Supplied

- Hotplate (with a solid plate and accurate temperature control up to 80°C)
- Variable volume micropipettes range 1 μ l to 200 μ l
- Water bath with accurate temperature control at 72°C
- Microcentrifuge tubes (0.5 ml)
- Fluorescence microscope (Please see Fluorescence Microscope Recommendation section)
- Plastic or glass coplin jars
- Forceps
- Fluorescence grade microscope lens immersion oil
- Bench top centrifuge

Fluorescence Microscope Recommendations

For optimal visualisation of the probe we recommend a 100 watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all three fluorophores simultaneously.

Sample preparation

The XYc kit is designed for use on cultured peripheral blood cells fixed in Carnoy's fixative and should be prepared according to the laboratory or institution guidelines.
The XYq kit is designed for use on cultured peripheral blood and bone marrow cells fixed in Carnoy's fixative and should be prepared according to the laboratory or institution guidelines.
Prepare blood and bone marrow metaphase spreads on microscope slides according to standard cytogenetic procedures.

FISH Protocol

Slide preparation

- Spot cell sample onto cleaned microscope slide
- Wash slide in 2 x SSC for 2 minutes at room temperature (RT: 20-25°C)
- Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 mins

Pre-Denaturation

- Remove probe from -20°C freezer and allow to warm to RT
- Ensure probe solution is uniform by repeated pipette mixing
- Remove 10 μ l of probe per test, place in a microcentrifuge tube
- Place probe, sample slide and 24x24mm glass coverslip to prewarm on a 37°C (+/- 1°C) hotplate for 10 mins
- Spot 10 μ l probe mixture onto the cell sample and carefully apply coverslip. Seal with rubber glue solution and allow to dry completely

Denaturation

- Denature sample and probe simultaneously by heating slide on a hotplate at 75°C (+/- 1°C) for 2 mins

Hybridisation

- Place slide in humid, lightproof container at 37°C (+/- 1°C) for 1 hour.

Post-Hybridisation Washes

- Remove coverslip and all traces of glue carefully
- Wash slide in 0.4 x SSC at 72°C (+/- 1°C) (pH 7.0) for 2 mins*
- Drain slide and wash in 2 x SSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds
- Drain the slide and apply 10 μ l of DAPI antifade
- Cover with a coverslip and allow colour to develop in the dark for 10 mins
- View with fluorescence microscope

* If background is observed, slides can be washed through 0.25xSSC at stage 12 instead of 0.4xSSC to increase stringency of washes.

Stability of Finished Slides

FISHED slides remain analysable for up to 1 month if stored in the dark at or below room temperature.

Procedural Recommendations

- The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
- The wash concentrations (stringency), pH and temperature are important, as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.

Expected Results

Chromosome X α -satellite (DXZ1) and Chromosome Y α -satellite (DYZ3)

- A diploid female sample should show a green fluorescence signal at the centromere of each X chromosome in 70-90% of cells analysed.
- A diploid male sample should show a green fluorescence signal at the centromere of the X chromosome together with a red fluorescence signal at the centromere of the Y chromosome in 70-90% of cells analysed.
- The probe will not detect constitutional or acquired structural sex chromosome abnormalities.

Chromosome X α -satellite (DXZ1) and Chromosome Y satellite III (DYZ1)

- A diploid female sample should show a green fluorescence signal at the centromere of each X chromosome in 70-90% of cells analysed.
- The probe identifies only the proportion of donor and recipient cells in bone marrow samples from recipients of opposite sex bone marrow transplantation. It does not distinguish between malignant and normal cells.
- Constitutional sex chromosome aneuploidy in the patient or donor will complicate signal enumeration and test interpretation when the probe is used for monitoring sex mismatched post-transplant bone marrow patients.

Customer Support

Please contact the Cytocell Sales and Marketing Department by telephone or e-mail. probes@cytocell.com

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Patents and Trademarks

Aquarius and CytoCELL are registered trademarks of CytoCELL Ltd.

Any cyanine dyes used in this Product are manufactured on behalf of Amersham Pharmacia Biotech Inc. under an exclusive license from Carnegie Mellon University and are covered by US Patent Number 5 268 486 and other patents pending. The Compound in this Product is manufactured by NEN Life Science Products, Inc. under US Patent Numbers 5 047 519 and 5 151 507. Use of the Product for commercial purposes is strictly forbidden without written permission from Amersham Pharmacia Biotech Inc. and NEN Life Science Products, Inc.

CytoCELL Ltd.
4 Technopark
Newmarket Road
Cambridge, CB5 8PB, UK.
T: +44(0)1223 294048
F: +44(0)1223 294986
E: probes@cytoCELL.com
W: www.cytoCELL.com

003/2010-06-28

DS#084/CE