## Instructions For Use

REF: LPU 017

## Neurofibromatosis Type 1 Probe with 17 pter Control

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Further information available at [www.cytocell.com](http://www.cytocell.com)

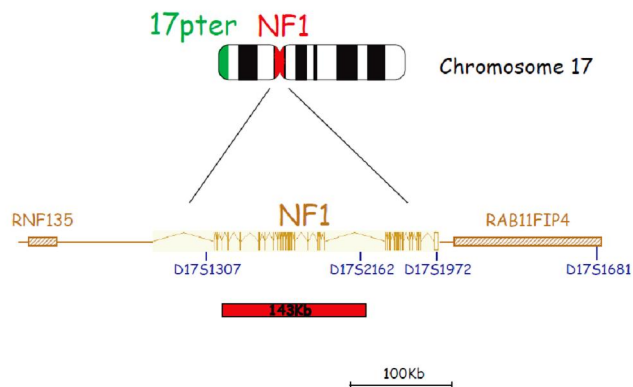
Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei of fixed cultured or uncultured cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Target DNA, after fixation and denaturation is available for annealing to a similarly denatured, fluorescently labelled DNA probe which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed by a series of rapid formamide-free stringent washes and the DNA counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

### Probe Specification

NF1 (17q11.2) Red  
17p Green

### Introduction

Neurofibromatosis type 1 (NF1) occurs in 1 in 3,000-4,000 people. It is a common autosomal dominant disorder characterised by neurofibromas, café-au-lait spots, freckles, Lisch nodules, bone deformities, learning disabilities, macrocephaly, short stature and predisposition to developing tumors such as myeloid malignancies, gliomas and pheochromocytomas<sup>1,2,3</sup>. In most cases the second copy of the NF1 gene becomes altered during a patient's lifetime. This event is most likely necessary to trigger tumour formation<sup>4</sup>. NF1 is caused by mutations of the tumour suppressor gene NF1, which spans approximately 280 kb and is located at the 17q11.2<sup>4,5</sup>. Mutations encompass both single nucleotide substitutions and large genomic deletions. Patients with deletions of the entire gene typically have a more severe presentation than those with intragenic mutations. Approximately 5-20% of patients with NF1 carry a heterozygous deletion and thus lack the NF1 gene and x 11 contiguous genes. Most interstitial fusions appear to directly link two regions of high sequence similarity that occur at distances of ~400 kb and 700 kb distal to the NF1 gene<sup>6</sup>.

The NF1 Region Probe labelled in red, is approximately 143 kb and contains the marker D17S2162. The accompanying 17 p specific probe, labelled in green, acts as a control probe. In the normal cell, there should be two red and two green signals (2R, 2G) or two yellow (2Y) signals, whilst a deletion of a probe target results in 1R, 2G or 0R, 2G or 1Y, 1G.

### Materials Provided

Probe: 50µl per vial (5 tests) or 100µl per vial (10 tests)

Amount of NF1 probe : 7.3 ng/test

Amount of 17p probe: 86.6 ng/test

The probe is provided premixed and ready to use in hybridisation solution (Formamide; Dextran Sulphate; SSC). The probe DNA is directly labelled: NF1 Probe with a red fluorophore (specificity to the Texas Red spectrum) and the 17p Control Probe with a green fluorophore (specificity to the FITC spectrum).

Counterstain: 150 µl per vial (15 tests)

The counterstain is DAPI antifade (ES: 0.125 µg/ml DAPI (4,6-diamidino-2-phenylindole)).

### Warnings and Precautions

- For *in vitro* diagnostic use. For professional use only.
- Wear gloves when handling DNA probes and DAPI counterstain.

- Probe mixtures contain formamide which is a teratogen ; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
- DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
- All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

### Storage and Handling

The Aquarius kit should be stored at 620°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

### Equipment Necessary but not Supplied

- Hotplate (with a solid plate and accurate temperature control up to 80°C)
- Variable volume micropipettes range 1 µl to 200 µl
- Water bath with accurate temperature control at 72°C
- Microcentrifuge tubes (0.5 ml)
- Fluorescence microscope (Please see Fluorescence Microscope Recommendation section)
- Plastic or glass coplin jars
- Forceps
- Fluorescence grade microscope lens immersion oil
- Bench top centrifuge

### Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100 watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all three fluorophores simultaneously.

### Sample Preparation

The kit is designed for use on cultured peripheral blood cells fixed in Carnoy's fixative and should be prepared according to the laboratory or institution guidelines. Prepare blood metaphase spreads on microscope slides according to standard cytogenetic procedures.

### FISH Protocol

#### Slide preparation

- Spot cell sample onto cleaned microscope slide.
- Immerse slide in 2 x SSC, pH 7.0 for 2 mins.
- Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 mins.

#### Pre-Denaturation

- Remove probe from -20°C freezer and allow to warm to RT.
- Ensure probe solution is uniform by repeated pipette mixing.
- Remove 10 µl of probe and place on cell sample slide, cover with a 24 x 24mm glass coverslip and seal with rubber solution glue.

#### Denaturation

- Place slide on to a 75°C (+/-1°C) hotplate and denature for 2 minutes.

#### Hybridisation:

- Hybridise slide overnight in a humid, lightproof container at 37°C (+/-1°C).

#### Post-Hybridisation Washes

- Remove coverslip and all traces of glue carefully.
- Wash slide in 0.4 x SSC at 72°C (+/-1°C) (pH 7.0) for 2 mins.
- Drain slide and wash in 2 x SSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds.
- Drain the slide and apply 10 µl of DAPI antifade.
- Cover with a coverslip and allow colour to develop in the dark for 10 mins.
- View with fluorescence microscope.

#### Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at or below room temperature.

#### Procedural Recommendations

- Baking or aging of slides is not recommended as it may reduce signal fluorescence.
- The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators, as these temperatures are critical for optimum product performance.
- The wash concentrations (stringency), pH and temperature are important, as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.

#### Customer Support

Please contact the Cytocell Sales and Marketing Department.

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#### Patents and Trademarks
Aquarius and Cytocell are registered trademarks of Cytocell Ltd.
**Cytocell Ltd.**